

ZEPF Drop-Control REF 46.007.02 Blade Holder DISMANTLING, CLEANING AND MAINTENANCE



Dear Customer!

We thank you very much for having chosen this high quality instrument from Helmut Zepf. To get the best from your instrument, please follow the instructions below:

DISMANTLING

The ZEPF Drop-Control Blade Holder has to be dismantled easily **before** cleaning, as shown.

⚠ Don't remove the spring

CLEANING

We recommend our „ZEPFcare SYSTEM“ in combination with a manual preparation.

Pre-Treatment: Use cleaning brush with nylon bristles when incrustations are heavy.

Cleaning: Manual or mechanical –
ATTENTION: Cleaning in a thermal disinfectant:
standing vertically, thread downwards.

Rinsing: Please use deionized water to avoid water spots.

Drying: Use compressed air or air-dry (manual preparation).

MAINTENANCE

Oiling with Medical White Oil, as shown.

ASSEMBLY

Assembly is made in reverse order to the dismantling.

1 DISMANTLING / MAINTENANCE

2

3

Oil after cleaning

⚠ Don't remove the spring

For detailed information about the complete process of instrument preparation and the coordinated product line, see our brochure:



Download

ZEPFcare
Optimum Cleaning

<http://www.zepf-dental.com/media/mediathek/protected/flyer/ZEPFCARE-E.pdf>

Cleaning Brushes with nylon bristles



85.903.01 double-ended, sterilizable, packing unit: 3 pieces



85.903.03 for aspirator Ø 1.5 mm



85.903.04 for aspirator Ø 3 mm and Ø 5 mm

Medical White Oil in a 50 ml dispensing bottle and in an Oil Pen, 12 ml



24.950.10



24.950.12