Selbstfüller-Wasserspritze mit Luer Lock Kanüle REF 19.750.00
Self-filling Water Syringe with Luer Lock Cannula REF 19.750.00

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Dear Customer!

We thank you very much for having chosen this high quality product from Helmut Zepf.
 To get the best from your instrument, please follow the instructions below:

IMPORTANT!

- The application syringe has to be cleaned and sterilized before use.
- Please make sure that this kind of syringe is only used by authorized qualified personnel.
- Cleaning and Sterilization: After use with a patient and prior to further use, the syringe has to be disassembled, cleaned and sterilized (in the autoclave at max. 134°C / 273°F).

COMPONENTS OF THE SELF-FILLING WATER SYRINGE

- Self-filling water syringe
- Luer Lock cannula REF 19.762.00 Spare part
- O-Ring REF 19.750.01 Spare part

HANDLING

- Connect the corresponding front part needed for the applications by means of the Luer-Lock-fitting onto the syringe and tighten it. The syringe is now ready for use.
- To avoid stiffness, the O-ring has to be lubricated before the first use and after each disinfection and sterilization by means of instrument oil.

REPROCESSING – SPECIAL FEATURES

- Remove obvious dirt particles from syringes immediately after utilization. It is not allowed to use fused agents or hot water (> 40°C) for cleaning as these agents lock residues and impede an effective purification process.
Attention: Maximum exposure time: 15 min.!
- **Manual pre-cleaning / cleaning:** In disassembled condition. Drill holes and thread turns must be flushed for at least 10 sec. at a pressure of 3.8 bars by means of a water pistol.
- **Cleaning according to DIN ISO 17664:2004:** For further information, see our brochure ZEPF|care Optimum Cleaning.
- **Drying:** Hollow spaces of the syringes must be dried by means of sterile compressed air.

19.750.00
 Ganzmetallspritze, selbstfüllend, mit 1 Kanüle (Luer-Lock-Ansatz)
 All-metal syringe, self-filling, with 1 cannula (Luer Lock connector)

19.762.00
 Kanüle mit Luer-Lock-Ansatz
 Cannula with Luer Lock connector

Download 🇩🇪 ▶
ZEPF|care
 Optimales
 Reinigen

Download 🇬🇧 ▶
ZEPF|care
 Optimum
 Cleaning