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Handling Instructions for Composite Instruments

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🇬🇧 Dear Customer!

We thank you very much for having chosen this high quality instrument from Helmut Zepf.

To maximize the life of your Composite Instruments, please follow the instructions below:

- Any contact of the instruments with amalgam has to be avoided (e.g. when inlays are placed or due to residues on gloves).
- Remove residues of material on the instrument immediately, still in the dental chair, e.g. by wiping with a cloth soaked in alcohol, in order to avoid drying-up. Thus, you avoid subsequent manual effort which might possibly lead to a counterproductive scratching of the instrument surface.
- Prior to sterilization, dry the instruments carefully with a soft cotton or microfiber cloth in order to avoid water spots.
- Do not use stainless steel brushes, brass brushes or abrasives for cleaning. To remove tenacious dirt, only use a firm toothbrush. Absolutely avoid any rough mechanical stress (abrasive detergents, e.g. detergent pads, cleaning cloths soiled by foreign particles, etc.).
- Avoid any aggressive cleaning agents (extremely acidic / basic, abrasive effect).
- If an ultrasonic cleaner is used, do not exceed the manufacturer's recommended ultrasonication period.
- After cleaning, rinse the instruments with clear or even better with demineralized water.
- If the instruments are cleaned in a disinfection bath, take care that the instruments' surfaces are not damaged by other instruments. A contact of the instrument parts during the cleaning process / during the sterilization has to be avoided (danger of contact corrosion).
- In thermal disinfection, we recommend to prepare these instruments in a washtray (see our sets 19.202.00, 26.120.00TI and 26.120.01TI) in order to avoid damages on the surface (washtray without instruments: 85.181.00 and 85.182.00).
- The instrument sets (washtray 19.202.00, 26.120.00TI and 26.120.01TI) can smoothly be sealed in sterile film in which they can be stored up to 6 weeks (acc. to DIN 58953 Part 7).

Thank you for following these instructions which will maximize the lifespan of your instruments.

Download 🇩🇪
ZEPF|care
 Optimales
 Reinigen

Download 🇬🇧
ZEPF|care
 Optimum
 Cleaning